

# OVERBED TABLE FINISHES & CARE

## OVERBED TABLES TOP & BASE FINISHES



### 175 OBT Top Finishes [xx]



AR ARIA



CO COCOBALA



SC SHAKER CHERRY



SO SOLAR OAK



TC TALAS CHERRY



WC WILD CHERRY



WM WINDSOR MAHOGANY

### 177-179 OBT Top Finishes [xx]



CL CANDLELIGHT



CP CHOCOLATE PEAR

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


HM HARD ROCK MAPLE



MO MEDIUM OAK

### 177-179 OBT Base Colors [y]



A ALMOND



G GRAPHITE GRAY



N NEVADA

### 175 Recommended Care & Cleaning<sup>RS</sup>



Clean the surface with a damp cloth or sponge and a mild soap or detergent. Rinse thoroughly with warm water and wipe dry.



Difficult stains such as coffee or tea can be removed using a mild household cleaner/detergent and a soft bristled brush, repeating as necessary.



If a stain persists, use a paste of baking soda and water and apply with a soft bristled brush. Light scrubbing for 10 to 20 strokes should remove most stains. Although baking soda is a low abrasive, excessive scrubbing or exerting too much force could damage the surface.



Stubborn stains that resist any of the above cleaning methods may require the use of undiluted household bleach or nail polish remover. Apply to the stain and let stand no longer than two minutes. Rinse thoroughly with warm water and wipe dry. May repeat if stain appears to be going away and the laminate color has not been affected.



After cleaning rinse surfaces using a water-dampened cloth, and dry thoroughly as some cleaning agents are corrosive by nature, and if not rinsed their residue can cause permanent damage

#### Recommended Household Cleaners<sup>RS</sup>

Windex® • Glass Plus® • Fantastik® • Mr. Clean® • Formula 409® • Isopropyl Alcohol

#### Warning

Prolonged exposure to bleach will cause discoloration. Always rinse surfaces after cleaning! Failure to rinse after cleaning can cause damage; even if a small amount of cleaning solution remains on the surface.

### 179-177 Recommended Care & Cleaning<sup>RS</sup>



**ALWAYS** rinse overbed table with a water-dampened cloth to remove any cleaning agent residue, and dry thoroughly



Recommend using a mild cleaning agent or detergent when cleaning overbed table. Some cleaning agents are corrosive by nature, and if not rinsed their residue can cause permanent damage



If other hospital-grade cleaning agents are used, follow the instructions provided by the manufacturer



After cleaning rinse surfaces using a water-dampened cloth, and dry thoroughly as some cleaning agents are corrosive by nature, and if not rinsed their residue can cause permanent damage

#### A Few Notes of Caution

- **Acidic or abrasive cleaners** can damage laminate surfaces; do not use them.
- **Hair, textile and food dyes** can cause permanent stains. If spilled, wipe it up immediately with dish-washing detergent or an all-purpose cleaner.
- **Steel wool and other abrasive pads** will damage the laminate face. Do not use them for cleaning.
- **Drain and Toilet bowl cleaners** contain harsh chemicals that can cause permanent damage. If spilled, wipe up immediately, wash surface with soapy water and rinse several times.

#### Color Variations

Swatches are digital reproductions. Perceived texture and coloring may vary on finished product. Monitor and printer settings will affect appearance of color.

<sup>R</sup> recommendations do not serve as a product guarantee, use manufacturer instructions  
<sup>S</sup> the term "safe" herein, pertains to surface effects and not in reference to human health

GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company

The most current and complete product information can be found on our website. Information contained herein is subject to change without notice.

GF2500042RevA25

published July 2025

Intensa, Inc. | 1810 S. Elm Street | High Point, NC 27260 | USA  
 +1 336.884.4096 | [intensa@grahamfield.com](mailto:intensa@grahamfield.com) | [intensa.net](http://intensa.net)

© 2015, GF Health Products, Inc. All Rights Reserved | Intensa, Inc. is a wholly owned subsidiary of GF Health Products, Inc. | Intensa is a registered trademark of GF Health Products, Inc. | Formula 409 is a registered trademark of The Clorox Company  
 Windex is a registered trademark of S. C. Johnson & Son, Inc. | Glass Plus are registered trademarks of Reckitt Benckiser LLC | Wilsonart is a registered trademark of Wilsonart LLC | Mr. Clean is a registered trademark of The Procter & Gamble Company

